

www.leoderm.kr

MAYA

MAYA EYES

High-quality tissue stimulator with PDRN



Salmon-derived PolyNucleotides
PN 0.2%

LEODERM


- **MAYA EYES ?**

Revitalize Your Cells. Restore Your Skin. Rediscover Youth.


MAYA EYES is a skin revitalizing treatment based on salmon-derived DNA.

Applied directly to the epidermis, it helps replenish moisture and vitality to damaged, tired-looking skin, promoting a healthier, firmer, and more radiant complexion.

- **MAYA EYES Key Ingredients**

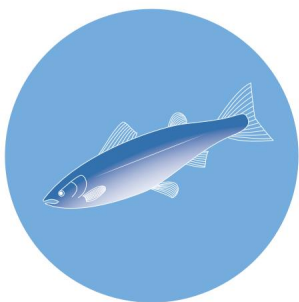

Salmon milt DNA

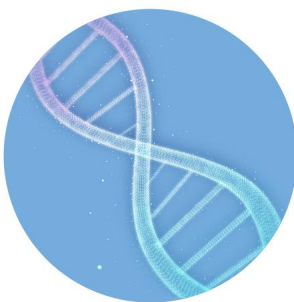


*Purified
Polynucleotide (PN)*



MAYA EYES

- **The Science Behind MAYA EYES**

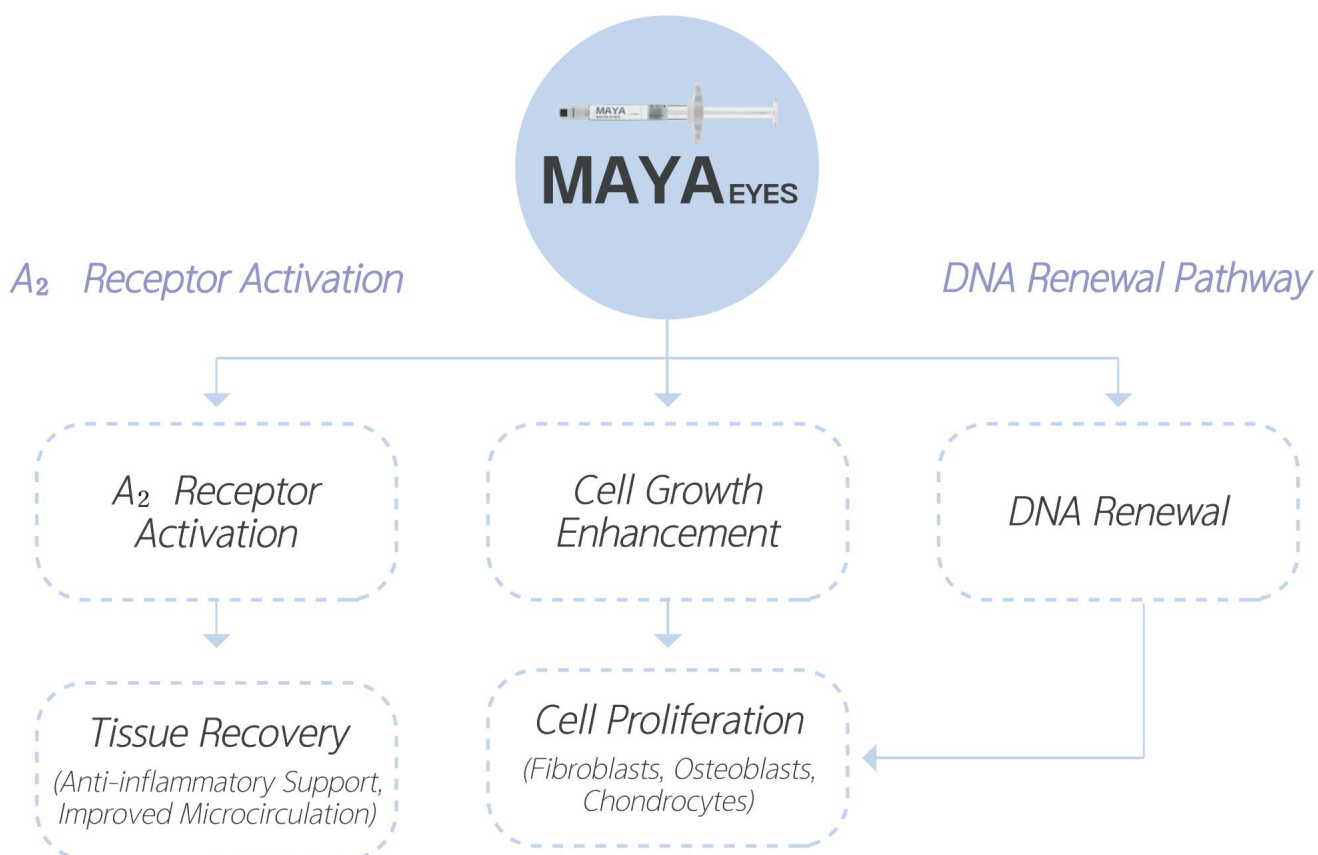

• ***MAYA EYES Benefits***

MAYA EYES harnesses the regenerative power of highly purified salmon-derived DNA to repair and revitalize damaged skin at the cellular level. Acting from the epidermis through the deep dermis, it stimulates tissue regeneration, enhances the skin's self-repair capacity, and restores optimal skin function. The strengthened skin barrier improves elasticity, firmness, hydration, and radiance, revealing smoother, healthier, and visibly younger-looking skin with long-lasting results.

• ***Cellular Regeneration & Tissue Restoration***

Radiance Enhancement

Cellular Regeneration

Skin Rejuvenation

Skin Barrier Restoration





Age-Defying

Elasticity Enhancement

Blemish Reduction

Wrinkle Improvement


• ***Target areas***

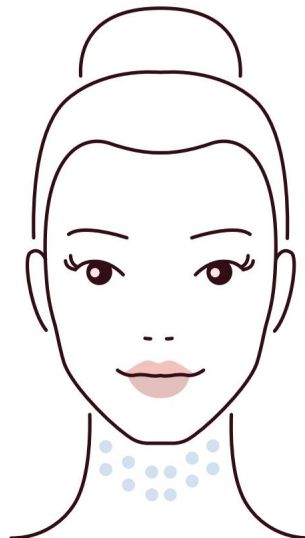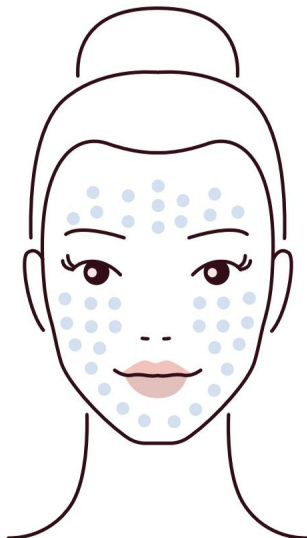

• Non-Clinical Study

Non-clinical studies demonstrated that MAYA EYES supports skin regeneration and promotes the recovery of damaged skin.

• Angiogenesis in Wound Healing

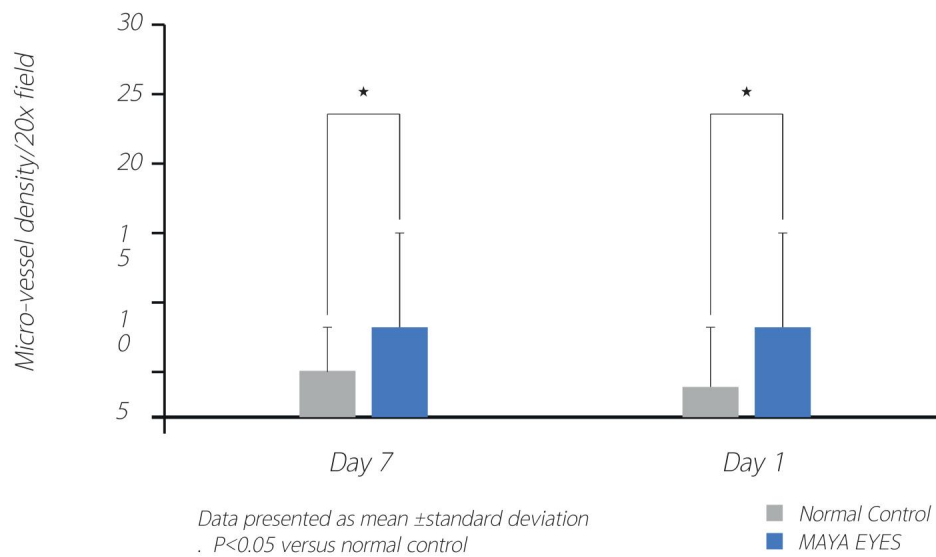

• Wound healing effects

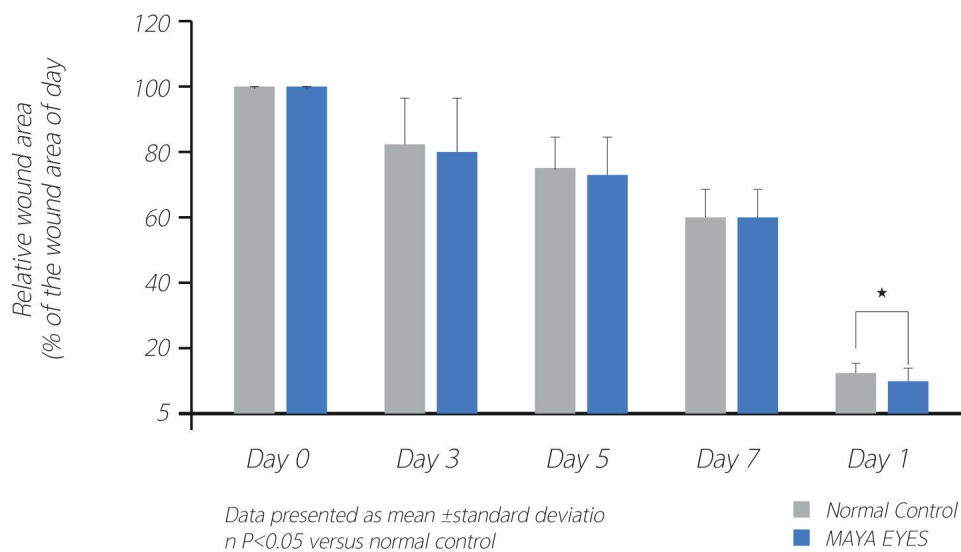


Figure2. Wound area

• Angiogenesis effects in a rat model of Hind limb ischemia.

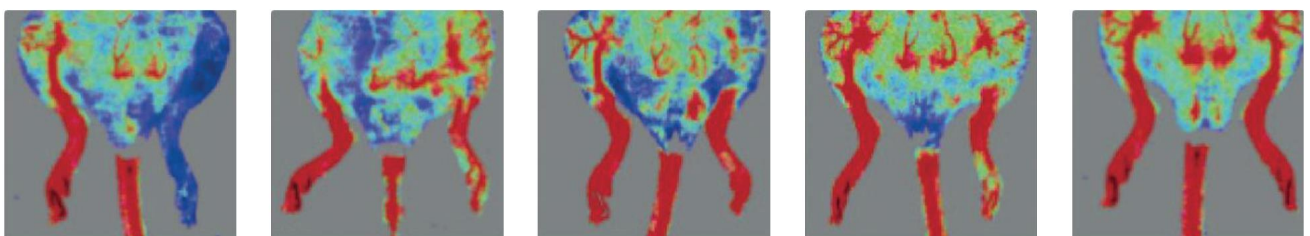


Figure3. Blood flow images (Day 21)

MAYA EYES

Advanced Hyaluronic Acid Complex

Restore • Rejuvenate • Radiate

Revitalizing Under-Eye Volume with Intensive Hydration



- **Package unit**

MAYA eyes 0.2% (Sodium DNA 2mg/ml)- 1ml

- **Treatment process**

MAYA eyes 's effects are maximized when 3 to 4 sessions are performed in 2 to 3 week intervals.

- After 3 to 5 days of treatment, the skin will feel soft and revitalized.
- After 2 to 4 weeks of treatment, there will be visible improvement of skin elasticity and fine wrinkles.
- After 4 to 6 weeks of treatment, the lifting and volume enhancing effects will become noticeable.

- **Recommended us**

When MAYA eyes is used in conjunction with Richesse (Hyaluronic Acid filler for skin hydration). It will boost the overall effectiveness of the product.

- **Precautions**

- Do not use on infected wounds (such as wounds with redness, pus, warmth, swelling, etc.).
- Suspend use immediately if you experience a rash, fever, allergic reactions, or symptoms of an infection while using the product, and consult your doctor or pharmacist for the most appropriate treatment.
- Use according to your doctor's instructions if you are hypersensitive to polynucleotide.
- Do not overuse.
- If you are pregnant or breastfeeding, or if you suspect or are planning to become pregnant, consult your doctor or pharmacist before use.

MAYA EYES

Advanced Hyaluronic Acid Complex



Restore. Rejuvenate. Radiate.

Advanced Cross-Linked HA Complex
Revitalizing Under-Eye Volume with Intensive Hydration

MAYA EYES

Restore Skin Quality. Reveal Natural Beauty.



MAYA EYES

High-quality tissue stimulator with PDRN

LEODERM

✉ info@leoderm.kr ☎ +8210-2858-8399 📞 +8210-2607-9983

📍 100, SUJEONGNAM-RO, SUJEONG-GU, SEONGNAM-SI, GYEONGGI-DO, South Korea.

www.leoderm.kr